



Max User Weight 318kg (50 stone)

Should you have a problem with this device, please quote the batch number attached to the frame.

The product is designed to aid persons with limited mobility / ability or whilst recovering from an injury or illness.

### **DESCRIPTION**

A heavy duty walking stick manufactured from chrome plated steel tube which is adjustable in height to suit most users.

### **INTENDED PURPOSE**

The stick is intended to aid the user in providing support when walking. It is designed so that the user may walk in as upright position as possible. The stick may be used individually or as a pair depending on the advice of the OT or physiotherapist.

Height Adjustable From 73-96cm / 29-38"

### **INSTRUCTIONS FOR USE**

Your dealer / OT should measure the individual user to determine which height would be the most suited. The user should have the necessary upper body strength to lift the stick forward whilst maintaining balance. The overall height can be adjusted by removing the 'C' clip, adjusting to the required height and reinserting the 'C' clip. Do not change the height of the stick without prior agreement of the person who undertook the initial assessment.

### **SAFETY PRECAUTIONS**

Always ensure you can confidently use the stick in your surrounding environment. Suitable for use on flat surfaces. Care should be taken when travelling over slopes. Not suitable for use on soft surfaces. Do not attempt to use on stairs or escalators when using a pair of sticks. Do not carry bags or other objects on the handles. Before use, ensure all fixings are secure and there is no excessive flex or movement in the structure. Do not use if any of the 'C' clips are damaged or loose.

### **CLEANING**

Do not use abrasive materials as this may damage the chrome surface. Use anti-bacterial wipes or domestic detergents and warm water to clean.