



**COBI REHAB®**  
The Positioning Company

## PRODUCT SHEET

# Bariatric 4-point Cane

**Strong and functional 4-point Cane specially designed to the bariatric user.**

4-point Cane can be used outdoors and indoors in institutions, care centers and in private homes. 4-point Cane is intended for users weighing up to 270 kg.



- Height adjustable
- Designed by Cobi Rehab
- Provide support and safety while walking
- Strong profile construction
- Weight capacity up to 270 kg



Cobi Rehab and Cobi Rehab's suppliers comply with UN conventions on child and forced labor and ILO 029, C132 and C182.

# Bariatric 4-point Cane

## General information

This product sheet contains important information about the use of 4-point Cane and the target group for the product. It also includes a range of specifications on 4-point Cane.

The most recent version of this product sheet can be found on our website. Check if the version you are reading now is the most recent.

If you need a product sheet that is written in a larger font size, the guide is available in PDF

format at [cobirehab.com](http://cobirehab.com), which can be scaled up to the required font size.

You can always contact Cobi Rehab on + 45 7025 2522 or [cobi@cobi.dk](mailto:cobi@cobi.dk) if you want more information about 4-point Cane.

Cobi Rehab reserves the right to change the product specifications without providing prior notice.

Product safety notices and product recalls can be viewed on the website.

## Use

4-point Cane is a cane specially designed to provide support and safety while walking for users weighing up to 270 kg.

4-point Cane is used at hospitals, care centers and in private homes. 4-point Cane should only be used on level ground and accessible terrain, indoors as well as outdoors.

## Contraindications

The user of 4-point Cane must have physical and cognitive resources to operate the cane in a safe manner.

## Settings

4-point Cane is supplied with a dual button height adjustment, a rubber handle, and a wrist strap.

The hand support is height adjustable by pressing the dual button..

## Cleaning and maintenance

4-point Cane can be cleaned with water and ordinary detergent.

## Disinfection

4-point Cane can be disinfected with ethanol 70-85% or a chlorine product 1000-1200 ppm.

## Long term storage

4-point Cane should be stored at temperatures between -10° and 50°C.

<b>Specifications</b>	
Maximum user weight	270 kg
Height adjustable	71-93 cm
Weight	1.5 kg
Base size	15 x 22 cm
Reflex in grip	Yes
Material	Chrome molybdenum steel/SAE41-30
Grip	TPR – Thermoplastic rubber
ISO - classification	ISO 12 03 16 - 03
Expected product service life	7 years
Risk class	I
UDI-DI	5740000102858
Basic UDI	5740000100010MV
The surface temperature of the 4-point Cane can increase or decrease when exposed to external sources of heat or cold.	

<b>Model</b>	<b>Height adjustable</b>	<b>Item number</b>
4-point Cane	710-930 mm	0201-000-000