

XXL REHAB[®]

MINIMAXX - FOLDING WHEELCHAIR

The XXL-Rehab Minimaxx folds up easily and is easy to bring along. The folding system is very easy to activate, and gives at the same time great support to the seat. The design of the folding system makes the chair rigid – when unfolded the chair is also much more stable than normal foldable chairs.

STW (Stiffness To Weight) was our goal. We used very light tubing technology as used on high end MTB and road bikes. This Minimaxx only weighs approx. 9% of its weight capacity and still provides a smooth drive thanks to the rigid folding system and light tubing.

The XXL-Rehab Minimaxx drives very well due to the position of the rear and front wheels. It is designed so the rear wheels carry most of the user's weight making it much easier for the user to push. The castor wheels are further in front as on standard chairs. This prevents the chair from tipping.

The low seat height makes it easy to access the chair and still propel it.

The push handles are designed so the attendant can easily use his/her body to turn and push the chair. The padded handles are at an angle that puts as little strain to the wrists and shoulders as possible.

The XXL-Rehab Minimaxx has disc brakes as an additional option. Little effort is required to brake the chair even with a full load. To the disc brake is also added a parking brake and the chair also has a wheels lock. Another option is the butterfly armrests. Butterfly armrests add more than 10 cm to the seating width.

The footrests are designed to carry 100 kg each! The footrests are also side/width adjustable. This enables users with legs aligned or with a wide leg spread to use the chair.

The very long armrests make it easy for the user to get in and out of the chair.

- Folding rigid frame
- Super easy to push
- Different armrests available, see page 25



Product video
Scan the code and see
the product in use

STRONGLY ENGINEERED AND FUNCTIONALLY DESIGNED
EXCLUSIVELY FOR BARIATRIC USERS!

SPECIFICATIONS

Maximum weight capacity is	325 kg
Seat width	56 / 61 / 66 / 71 cm
Total width	Seat width + 22 cm
Total width folded	35 cm
Total height	103 cm
Total length without footrests	70 cm
Seat depth	46 cm
Seat height	43 cm
Backrest height	51 cm
Weight	Approx. 27 kg
ISO – classification	ISO 12 22 03 – 08

ORDERING

Model	Seat width cm	Seat depth cm	Order No
XXL-Rehab Minimaxx Standard incl. footrests	56	46	0100-056-000
XXL-Rehab Minimaxx Standard incl. footrests	61	46	0100-061-000
XXL-Rehab Minimaxx Standard incl. footrests	66	46	0100-066-000
XXL-Rehab Minimaxx Standard incl. footrests	71	46	0100-071-000
XXL-Rehab Minimaxx with disc brakes, incl. footrests	56	46	0109-056-000
XXL-Rehab Minimaxx with disc brakes, incl. footrests	61	46	0109-061-000
XXL-Rehab Minimaxx with disc brakes, incl. footrests	66	46	0109-066-000
XXL-Rehab Minimaxx with disc brakes, incl. footrests	71	46	0109-071-000
Accessories			
Butterfly armrests, pair			0100-099-005
Sport armrests, pair			0100-099-006
Table			0100-099-001
Cane holder for 1 cane			0100-099-022
IV holder and O2 holder, diameter 10 cm			0100-099-020
Calf support / amputation support, right			0100-099-015
Calf support / amputation support, left			0100-099-016
Footrest swing-away, right			0100-099-010
Footrest swing-away, left			0100-099-011




Disc brakes



Table



IV holder and O2 holder

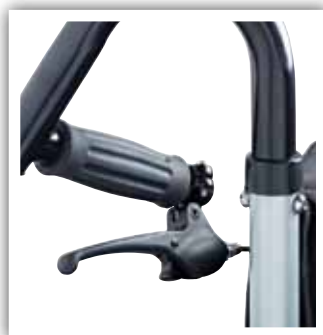

Hand brake operated by attendant



Calf support / amputation support